SHIGA TOXIN QUIK CHEK™

A Rapid Membrane Enzyme Immunoassay for the Simultaneous
Qualitative Detection and Differentiation of Shiga toxin 1 and
Shiga toxin 2 in Human Fecal Specimens and
Cultures Derived from Fecal Specimens

Catalog No. T30625 (25 Tests)

IVD *In Vitro* Diagnostic Medical Device

For Canadian Users: For Laboratory Use Only

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SHIGA TOXIN QUIK CHEK™

INTENDED USE

The *SHIGA TOXIN QUIK CHEK™* test is a rapid membrane enzyme immunoassay for the simultaneous qualitative detection and differentiation of Shiga toxin 1 (Stx1) and Shiga toxin 2 (Stx2) in a single test device. It is intended for use with human fecal samples from patients with gastrointestinal symptoms to aid in the diagnosis of disease caused by Shiga toxin producing *Escherichia coli* (STEC). It may be used with fecal specimens, or broth or plate cultures derived from fecal specimens. The test results should be considered in conjunction with the patient history.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a physician.

EXPLANATION

Shiga toxin producing *Escherichia coli* (STEC) were first described by O' Brien, *et.al.* after discovering that *E. coli* culture supernatant, which was cytotoxic to HeLa and Vero cells, could be neutralized by rabbit anti-Shiga toxin antibodies (1). STEC cause foodborne and waterborne diarrheal disease worldwide which, if left undiagnosed, can progress to hemorrhagic colitis and/or hemolytic uremic syndrome (HUS) (2, 3). Since certain treatments and medications can increase the risk of HUS (4), prompt detection is necessary to prevent outbreaks and secondary transmission (5-9). STEC strain O157:H7 has historically been the focus of attention in the United States since first isolated from undercooked hamburgers (3, 10), causing an estimated 73,000 illnesses annually (11). However, STEC infections caused by non-O157 strains have become more prevalent in recent years, both in the United States as well as abroad (12-16, 28). O157:H7 infections are routinely diagnosed by culture of fecal samples on selective media (17, 18), but this methodology allows non-O157 STEC strains to go undetected. STEC produce either one or both Shiga toxins (Stx1 and/or Stx2), both potent cytotoxins (19, 20). Isolates producing only Stx2 have been attributed to higher incidence rates of HUS (18, 21-23). Shiga toxins can be detected by tissue culture assay (24), but this method is both time consuming and labor intensive. By detecting the toxins, the *SHIGA TOXIN QUIK CHEK™* test can detect STEC present in fecal samples or culture, regardless of the serotype or other virulence factors (25).

PRINCIPLE OF THE TEST

The *SHIGA TOXIN QUIK CHEK™* test utilizes specific antibodies against Stx1 and Stx2. The *Membrane Device* contains a *Reaction Window* with three vertical lines of immobilized antibodies. The "1" test line contains monoclonal antibodies against Stx1. The control line ("C") is a dotted line that contains anti-horseradish peroxidase (HRP) antibodies. The "2" test line contains monoclonal antibodies against Stx2. The *Conjugate* consists of antibodies to Stx1 and Stx2 coupled to horseradish peroxidase. To perform the test, the sample is added to a tube containing a mixture of *Diluent* and *Conjugate*. The diluted sample-conjugate mixture is added to the *Sample Well* and the device is allowed to incubate at room temperature for 15 minutes. During the incubation, any Stx1 and/or Stx2 present in the sample binds to the antibody-peroxidase conjugates. The toxin-antibody-peroxidase complexes migrate through a filter pad to a membrane where they are captured by the immobilized Stx1 and Stx2 specific monoclonal antibodies in the test lines. The *Reaction Window* is subsequently washed with *Wash Buffer*, followed by the addition of *Substrate*. After a 10 minute incubation period, the *Reaction Window* is examined visually for the appearance of vertical blue lines on the "1" and "2" sides of the *Reaction Window*. A blue line on the "1" side of the *Reaction Window* is a positive result indicating the presence of Stx1. A blue line on the "2" side of the *Reaction Window* is a positive result indicating the presence of Stx2. A positive "C" reaction, indicated by a vertical dotted blue line under the "C" portion of the *Reaction Window*, confirms that the test is working properly, the procedure was followed, and the results are valid.

MATERIALS PROVIDED

- MEM | DEV** **Membrane Devices** – each pouch contains 1 device
- DIL | SPE** **Diluent (22 mL)** – Buffered protein solution with graduated dropper assembly*
H412 - Harmful to aquatic life with long lasting effects.
- WASH | REAG** **Wash Buffer (12 mL)** – Buffered solution with graduated dropper assembly*
- SUBS | REAG** **Substrate (3.5 mL)** – Solution containing tetramethylbenzidine
- CONJ | ENZ** **Conjugate (2.5 mL)** – Antibodies specific for Stx1 and Stx2 coupled to horseradish peroxidase in a buffered protein solution*
- CONTROL | +** **Positive Control (1 mL)** – Antigen in a buffered protein solution*
- Disposable plastic transfer pipettes** – graduated at 25 µL, 100 µL, 200 µL, 300 µL, 400 µL and 500 µL
- IVD** *In Vitro* Diagnostic Medical Device

*contains 0.05% ProClin® 300

Signal Word: Warning

H317: May cause an allergic skin reaction

P261, P272, P280, P302, P352, P333, P313, P321, P362, P363, P364, P501

**MATERIALS AND EQUIPMENT REQUIRED BUT NOT PROVIDED**

Small test tubes (e.g., plastic Eppendorf tubes or glass tubes)

Applicator sticks or swabs

Timer

Vortex mixer

Disposable gloves for handling fecal samples

Pipettor and tips

SHELF LIFE AND STORAGE

The expiration date of the kit is given on the label. Expiration dates for each component are listed on the individual labels. The kit should be stored between 2°C and 8°C. The kit containing the reagents with designated shelf life should be stored between 2°C and 8°C and should be returned to the refrigerator as soon as possible after use.

PRECAUTIONS

1. Rx Only - Prescription Only
2. For *in vitro* diagnostic use. For professional use only.
3. Each component in the kit should be inspected for any signs of leakage. Upon arrival, inspect the kit to ensure that components are not frozen or warm to the touch due to improper shipping conditions.
4. The *Substrate* reagent should be colorless. If the *Substrate* reagent changes to a dark blue/violet color, discard and call Technical Services for a replacement.
5. Reagents from different kits should not be mixed or interchanged. Do not use a kit past the expiration date.
6. Caps, tips and dropper assemblies are color-coded; do NOT mix or interchange!
7. Bring all components to ROOM TEMPERATURE BEFORE USE!
8. Do not freeze the reagents. The kit should be stored between 2°C and 8°C.
9. The pouch containing the *Membrane Device* should be at room temperature before opening. Keep the membrane devices dry before use.
10. Hold reagent bottles vertically to dispense reagents to ensure consistent drop size and correct volume.
11. Microbial contamination of reagents may decrease the accuracy of the assay. Avoid microbial contamination of reagents by using sterile disposable pipettes if removing aliquots from reagent bottles.
12. Membrane devices cannot be reused.
13. The test has been optimized for sensitivity and specificity. Alterations of the specified procedure and/or test conditions may affect the sensitivity and specificity of the test. Do not deviate from the specified procedure.
14. The validity of the test results using the *SHIGA TOXIN QUIK CHEK™* test is dependent upon the proper reaction of the internal and external controls. See the Quality Control section.

4

15. Specimens and used membrane devices should be handled and disposed of as potential biohazards after use. Wear disposable gloves when doing the test.
16. Fecal specimens may contain potentially infectious agents and should be handled at “Biosafety Level 2” as recommended in the CDC/NIH Manual “Biosafety in Microbiological and Biomedical Laboratories.”
17. The *Conjugate*, *Diluent*, *Positive Control*, and *Wash Buffer* reagents contain 0.05% ProClin® 300 as a preservative. Although the concentration is low, ProClin® 300 is known to be harmful (skin sensitization may occur). If skin sensitization/irritation or rash occurs, get medical advice/attention. Take off contaminated clothing and wash it before reuse. Handle reagents according to existing regulations for laboratory safety and good laboratory practice. Safety Data Sheets for this product are available upon request, contact technical support.
18. Follow your national, regional, and local ordinances accordingly for waste disposal regulations.

COLLECTION, HANDLING, AND STORAGE OF FECAL SPECIMENS

CDC guidelines for STEC diagnostic testing recommend testing specimens as soon as received by the laboratory.

Acceptable Sample Types	Do Not Use
Fresh Fecal Specimens	Fecal specimens in Formalin-based fixative (e.g. sodium acetate formalin, 10% formalin)
Specimens in transport media (e.g. Cary Blair, C&S)	Fecal specimens in alcohol-based fixative (e.g. polyvinyl alcohol)
Frozen Fecal Specimens (frozen undiluted or frozen in transport media)	
Gram Negative or MacConkey broth cultures grown from an acceptable sample type	
Bacterial cultures from a SMAC, CT-SMAC, or CHROMagar® O157 plate, grown from any acceptable sample type	

1. **Specimen Handling for Direct Fecal Testing –**
 - a. Fresh specimens should be tested as soon as possible after receipt. If testing cannot be performed upon receipt, samples may be stored between 2° and 8°C or frozen ($\leq -10^{\circ}\text{C}$) for up to 14 days from sample receipt.
 - b. Specimens in transport media (C&S or Cary Blair) can be stored between 2° and 8°C or frozen ($\leq -10^{\circ}\text{C}$) for up to 14 days from sample receipt.
2. **Specimen Handling for Broth or Plate Method –**
 - a. Specimens should be stored between 2° and 8°C and cultured as soon as possible after receipt. If cultures cannot be started within 2 hours of sample receipt, samples may be stored frozen ($\leq -10^{\circ}\text{C}$) for up to 14 days from sample receipt.
 - b. Specimens in transport media (C&S or Cary Blair) can be stored between 2° and 8°C for up to 5 days.
3. Make sure that specimens are thoroughly mixed before performing the assay.
4. Repeated freeze/thaw cycles should be avoided. If using frozen specimens, thaw at room temperature.
5. Storing fecal specimens in the *Diluent* is NOT recommended.
6. Do not allow the samples to remain in the *Diluent/Conjugate* mixture for more than two hours.

SPECIMEN PREPARATION

A. Direct Fecal Specimen Testing (for fresh specimens and samples in transport media)

1. Mix all specimens thoroughly regardless of consistency - it is essential that the specimens be evenly suspended before sampling.
2. Continue to **TEST PROCEDURE**.

B. Broth Method (for fresh specimens and samples in transport media)

1. Mix all specimens thoroughly regardless of consistency - it is essential that the specimens be evenly suspended before inoculating the broth.
 - a. **Liquid/Semi-solid specimens** - transfer 25 μ L of specimen into a culture tube containing 5 mL of MacConkey or 8 mL of Gram-Negative (GN) broth.
 - b. **Formed/Solid specimens** - transfer a small portion (approximately 2 mm diameter, the equivalent of 25 μ L) of the specimen into a culture tube containing 5 mL of MacConkey or 8 mL of GN broth.
 - c. **Fecal specimens in Cary Blair or C&S transport media** - transfer 100 μ L of the preserved specimen to a culture tube containing 5 mL of MacConkey or 8 mL of GN broth.
2. Loosely cap the inoculated broth tubes and incubate for 16-24 hours between 35° and 39°C.
3. Examine the tube for growth. If there is no growth, do not proceed with testing. Instead, inoculate another tube of broth with either the same fecal specimen or a fresh specimen from the same patient. Alternatively, the selective plate method (see “C” below) or direct fecal specimen testing method (see “A” above) may be used.
4. Continue to **TEST PROCEDURE**.


C. Plate Method (for fresh specimens and samples in transport media)

1. Mix all specimens thoroughly regardless of consistency - it is essential that the specimens be evenly suspended before inoculating the plate. Use a swab to sample the specimen, and then spread on a SMAC, CT-SMAC, or CHROMagar® O157 plate. *NOTE: CT-SMAC and CHROMagar® O157 are more selective than SMAC plates and may inhibit the growth of non-O157 STEC.*
2. Incubate the plates for 16-24 hours between 35°C and 39°C.
3. Examine the plate for growth. If there is no growth, do not proceed with testing. Instead, inoculate another plate with either the same fecal specimen or a fresh specimen from the same patient. Alternatively, the broth method (see “B” above) or direct fecal specimen testing method (see “A” above) may be used.
4. Continue to **TEST PROCEDURE**.

TEST PROCEDURE


1. Bring all reagents and the required number of devices to room temperature before use.
2. **Set up and label one small test tube for each sample, and optional external controls as necessary.**
3. **Add Diluent to each tube using the black graduated dropper assembly.**

Sample Type	Volume of Diluent
Fecal Specimen (unpreserved) Plate Culture External Controls	750 μ L (2nd graduation from tip)
Broth Culture Transport Media Specimen	650 μ L (1st graduation from tip)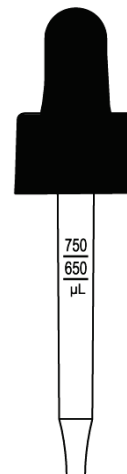

- 6
4. **Add one drop of *Conjugate* (red capped bottle) to each tube.**
Be attentive to the total assay time when testing more than one fecal specimen. The *Diluent* and *Conjugate* should be added to all tubes prior to adding the specimens.
 5. Obtain one disposable plastic transfer pipette (supplied with the kit) for each sample – the pipettes have raised graduations at 25 μL , 100 μL , 200 μL , 300 μL , 400 μL and 500 μL .

Graduated Transfer Pipette: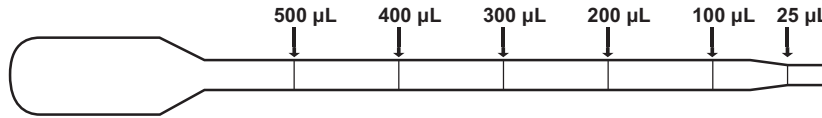


6. **Mix all specimens and cultures thoroughly regardless of consistency- it is essential that the samples be evenly suspended before sampling. Add the required amount of specimen or culture to the tube.**
Direct fecal testing - Liquid/Semi-solid specimens - Using a transfer pipette, transfer 25 μL of specimen into the *Diluent/Conjugate* mixture. Use the same transfer pipette to mix the diluted specimen.
Direct fecal testing - Formed/Solid specimens – Care must be taken to add the correct amount of formed feces to the sample mixture. Mix the specimen thoroughly using a wooden applicator stick and transfer a small portion (approximately 2 mm diameter, the equivalent of 25 μL) of the specimen into the *Diluent/Conjugate* mixture. Emulsify the specimen using the applicator stick.

Broth cultures and Fecal specimens in transport media (Cary Blair or C&S) - Using a transfer pipette, transfer 100 μL of specimen into the *Diluent/Conjugate* mixture.

Plate cultures - Sweep through a confluent area of the plate several times with a swab, then mix the swab in the *Diluent/Conjugate* mixture. Rotate the swab against the inside of the test tube several times to release the sample from the swab. **Remove the swab, mix well and allow to incubate for 30 minutes at room temperature before proceeding with Step 8 of the TEST PROCEDURE.**

NOTE: Transferring too little sample, or failure to mix and completely suspend the sample in the Diluent mixture, may result in a false-negative test result. The addition of too much sample may cause invalid results due to restricted flow.

7. **Optional External Controls:**
External Positive Control - add one drop of *Positive Control* (gray-capped bottle) to the appropriate test tube.
External Negative Control - add 25 μL *Diluent* to the appropriate test tube.
8. Close each tube of diluted sample or control and mix thoroughly. Proper mixing can be achieved by vortexing or inverting the tube several times. Once a sample or control has been diluted in the *Diluent/Conjugate* mixture, it may be incubated at room temperature for any period of time up to 2 hours prior to addition to the *Membrane Device*.
9. Obtain one *Membrane Device* for each sample and optional external positive or negative control as necessary. The foil bags containing the devices should be brought to room temperature before opening. Label each device appropriately and orient it on a flat surface so the "SHIGA TOXIN" print is at the bottom of the device, and the small *Sample Well* is located in the top right corner of the device.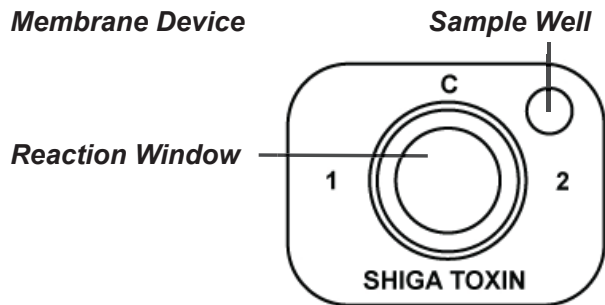

10. Make sure that each sample is thoroughly mixed before addition to the *Membrane Device*. **Using a new transfer pipette, transfer 500 μ L from each tube into the Sample Well (smaller hole in the top right corner of the device) of a *Membrane Device*, making certain to expel the liquid sample onto the wicking pad inside of the *Membrane Device*.** When loading the sample into the sample well, make sure that the tip of the transfer pipette is angled towards the *Reaction Window* (larger hole in the middle of the device).
11. **Incubate the device at room temperature for 15 minutes** – the sample will wick through the device and a wet area will spread across the *Reaction Window*. The 15-minute incubation step begins after the last diluted sample-conjugate mixture has been transferred to the final *Membrane Device*.
NOTE FOR SAMPLES THAT FAIL TO MIGRATE:
Occasionally, a diluted sample fails to migrate properly and the Reaction Window does not fully wet. If the Reaction Window does not appear to be completely wet within 5 minutes of adding the sample to the Sample Well, then add 100 μ L (4 drops) of Diluent to the Sample Well and wait an additional 5 minutes (for a total of 20 minutes).
12. **After the incubation, add 300 μ L of Wash Buffer to the Reaction Window using the graduated white dropper assembly (or equivalent).** Allow the *Wash Buffer* to flow through the *Reaction Window* membrane and be absorbed completely.
13. **Add 2 drops of Substrate (white-capped bottle) to the Reaction Window.** Read visually and record results after 10 minutes.

INTERPRETATION OF RESULTS

1. Interpretation of the test is most reliable when the device is read immediately at the end of the 10 minute reaction period. A test cannot be interpreted before that time. Read the device at a normal working distance in a well-lit area. View with a line of vision directly over the device.
2. Observe device for the appearance of a dotted blue line in the middle of the *Reaction Window* representing the internal positive control. The appearance of any control dot(s) represents a valid internal control.
3. Observe device for the appearance of blue lines on the “1” and “2” sides of the *Reaction Window* representing the test lines. The lines may appear faint to dark in intensity. An obvious partial line is interpreted as a valid line. Do not interpret membrane discoloration as a positive result.
Positive Stx1 (“1”) Result: The blue “1” line and the dotted blue control line below “C” are visible (Figure 1a). A positive result indicates the presence of Stx1.
Positive Stx2 (“2”) Result: The blue “2” line and the dotted blue control line below “C” are visible (Figure 1b). A positive result indicates the presence of Stx2.
Positive Stx1 and Stx2 (“1” and “2”) Result: Both the blue “1” and blue “2” lines are visible as well as and the dotted blue control line below “C” (Figure 1c). A positive result indicates the presence of both Stx1 and Stx2.
4. **Negative Result:** A single blue dotted line is visible in the middle of the *Reaction Window*, below the “C” and no test lines are visible on the “1” side or the “2” side of the *Reaction Window* (Figure 1d). A negative result on “1” side indicates that Stx1 is either absent in the sample or is below the detection limit of the test. A negative result on “2” side indicates that Stx2 is either absent in the sample or is below the detection limit of the test.

5. **Invalid Result:** No lines are visible in the *Reaction Window* (Figure 1e) or a blue dotted line is not present below the “C” at the completion of the reaction period (Figures 1f, 1g, 1h).
6. A positive result in the *SHIGA TOXIN QUIK CHEK™* confirms the presence of Stx1 and/or Stx2 in the sample; a negative result indicates the absence of toxin or insufficient levels of toxin for detection.

Note: Due to the epidemiological importance of obtaining Shiga toxin positive bacterial isolates, it is recommended that all toxin positive samples undergo bacterial culture to isolate the toxin producing organisms. It is suggested that laboratories perform bacterial culture on all positive samples or coordinate the process with their local & state health laboratories in the United States.

FIGURE 1: SHIGA TOXIN QUIK CHEK™ INTERPRETATION OF RESULTS

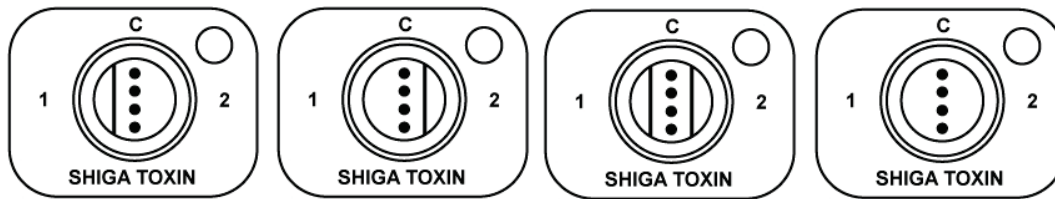
Figure 1a
Positive Stx1 Result

Figure 1b
Positive Stx2 Result

Figure 1c
Positive Stx1 and
Stx2 Result

Figure 1d
Negative Result

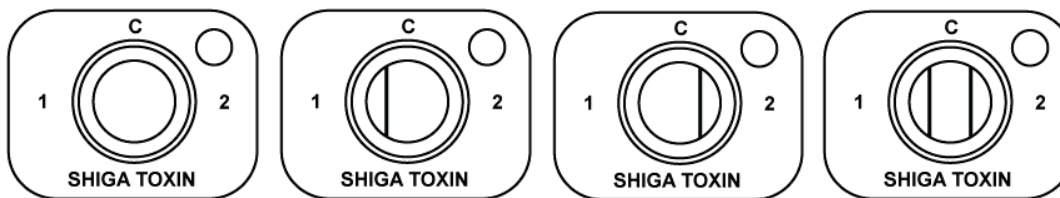
Figure 1e
Invalid Result

Figure 1f
Invalid Result

Figure 1g
Invalid Result

Figure 1h
Invalid Result

QUALITY CONTROL

Internal: A dotted blue line must be visible in the middle of the *Reaction Window*, below the “C” on every *Membrane Device* that is tested. The appearance of the blue control dotted line confirms that the sample and reagents were added correctly, that the reagents were active at the time of performing the assay, and that the sample migrated properly through the *Membrane Device*. A clear background in the result area is considered an internal negative control. If the test has been performed correctly and reagents are working properly, the background will be white to light blue to give a discernible result.

External: The reactivity of the *SHIGA TOXIN QUIK CHEK™* kit should be verified upon receipt using the *Positive Control* and negative control (*Diluent*). The *Positive Control* is supplied with the kit (gray-capped bottle). The *Positive Control* confirms the reactivity of the other reagents associated with the assay, and is not intended to ensure precision at the analytical assay cut-off. *Diluent* is used for the negative control. Additional tests can be performed with the controls to meet the requirements of local, state and/or federal regulations and/or accrediting organizations.

LIMITATIONS

1. The *SHIGA TOXIN QUIK CHEK™* test is used to detect Stx1 and Stx2 in fecal specimens and cultures derived from fecal specimens. The test confirms the presence of Stx1 and/or Stx2 in the sample, and this information should be taken under consideration by the physician in light of the clinical history and physical examination of the patient.

2. A negative test result does not preclude the possibility of the presence of Shiga toxins in the specimen which may occur if the level of antigen is below the detection limit of the test.
3. The *SHIGA TOXIN QUIK CHEK*[™] test is qualitative. The intensity of the color should not be interpreted quantitatively.
4. The toxin produced by *Shigella dysenteriae* is nearly identical to Stx1 produced by *E. coli* (26), and if present at detectable levels, will give a positive result on the "1" side of the *Reaction Window*.

EXPECTED VALUES

The *SHIGA TOXIN QUIK CHEK*[™] test detects the presence of Stx1 and Stx2. Expected values for a particular population should be established by each laboratory. The positivity rate may be dependent upon a number of factors including geography, process of specimen collection, handling and transport, and patient age.

Shiga toxin *E. coli* is the source of an estimated 110,000 cases (0.04% of the population) of foodborne illness annually in the United States (11). Reported incidence rates in fecal samples submitted for testing range from 0% - 4.1% (18) and vary depending upon the season, geographical location, and patient population, with higher incidence rates seen in the summer months and in preschool-aged children and the elderly (27). A positive result in the *SHIGA TOXIN QUIK CHEK*[™] confirms the presence of Stx1 and/or Stx2 in the sample; a negative result indicates the absence of toxin or insufficient levels of toxin for detection.

PERFORMANCE CHARACTERISTICS

The performance of the *SHIGA TOXIN QUIK CHEK*[™] test was evaluated at 3 independent sites. A summary of overall performance at the 3 sites follows.

Direct Fecal Testing

The performance of the *SHIGA TOXIN QUIK CHEK*[™] (STQC) test was compared to the Vero Cell Cytotoxin Assay (with neutralization), considered the clinical reference standard (gold standard) and included 873 fresh and 14 frozen samples. Age and sex information was available for 878 patients. Of the 878 patients, 8% were ≤ 18 years and 59.8% were females and 40.2% were males. The following tables show a summary of the clinical performance of the Stx1 portion and the Stx2 portion of the *SHIGA TOXIN QUIK CHEK*[™] test at all 3 sites. The results show that the Stx1 portion exhibited a sensitivity of 98.0%, a specificity of 99.8%, and an overall correlation of 99.7% with cytotoxin assay. The Stx2 portion exhibited a sensitivity of 98.0%, a specificity of 100%, and an overall correlation of 99.9% with cytotoxin assay.

Direct Fecal Testing Results

	Vero Cell Cytotoxin Assay	
	Stx1 +	Stx1 -
n = 887		
STQC Stx1 +	48	2
STQC Stx1 -	1	836

	Vero Cell Cytotoxin Assay	
	Stx2 +	Stx2 -
n = 887		
STQC Stx2 +	48	0
STQC Stx2 -	1	838

		95% Confidence Interval
Sensitivity	98.0%	87.8 - 99.9%
Specificity	99.8%	99.0 - 99.9%
Correlation	99.7%	99.7 - 99.7%

		95% Confidence Interval
Sensitivity	98.0%	87.8 - 99.9%
Specificity	100%	99.4 - 99.9%
Correlation	99.9%	100 - 100%

Broth Cultures

The performance of the *SHIGA TOXIN QUIK CHEK*[™] test using overnight broth cultures (GN or MacConkey broth) from fecal specimens was compared to the Vero Cell Cytotoxin Assay (with neutralization). The following tables show a summary of the clinical performance of the Stx1 portion and the Stx2 portion of the *SHIGA TOXIN QUIK CHEK*[™] test. The results show that the Stx1 portion exhibited a sensitivity of 100%, a specificity of 99.5%, and an overall correlation of 99.5% with cytotoxin assay. The Stx2 portion exhibited a sensitivity of 95.7%, a specificity of 99.9%, and an overall correlation of 99.6% with cytotoxin assay.

Broth Culture Testing Results

n = 770	Vero Cell Cytotoxin Assay	
	Stx1 +	Stx1 -
STQC Stx1 +	42	4
STQC Stx1 -	0	742

n = 770	Vero Cell Cytotoxin Assay	
	Stx2 +	Stx2 -
STQC Stx2 +	45	1
STQC Stx2 -	2	722

		95% Confidence Interval
Sensitivity	100%	89.6 - 100%
Specificity	99.5%	98.5 - 99.8%
Correlation	99.5%	99.5 - 99.5%

		95% Confidence Interval
Sensitivity	95.7%	84.3 - 99.3%
Specificity	99.9%	99.1 - 100%
Correlation	99.6%	99.6 - 99.6%

REPRODUCIBILITY

The reproducibility of the *SHIGA TOXIN QUIK CHEK*[™] test was determined using 12 fecal specimens that were coded to prevent their identification during testing. Testing was performed at 2 independent laboratories and on-site at TECHLAB, Inc. The samples were tested, twice a day over a 5-day period by multiple technicians at each site using 2 different kit lots. A positive and negative control was run with each panel of the masked samples. The results from each laboratory were submitted to TECHLAB, Inc. and compared with in-house results. The results were consistent among the different locations, and exhibited a correlation of 100%. The samples produced the expected results 100% of the time.

CROSS REACTIVITY

The *SHIGA TOXIN QUIK CHEK*[™] test was evaluated for cross-reactivity with the bacterial and viral strains listed below. None of the strains were shown to interfere with the performance *SHIGA TOXIN QUIK CHEK*[™] test.

<i>Aeromonas hydrophila</i>	<i>Campylobacter coli</i>	<i>Campylobacter fetus</i>
<i>Campylobacter jejuni</i>	<i>Candida albicans</i>	<i>Citrobacter freundii</i>
<i>Clostridium difficile</i>	<i>Clostridium perfringens</i>	<i>Enterobacter cloacae</i>
<i>Enterococcus faecalis</i>	<i>Escherichia coli</i> (non-toxigenic)	
<i>Escherichia coli</i> O157:H7 (non-toxigenic)		<i>Escherichia coli</i> EIEC (enteroinvasive)
<i>Escherichia coli</i> EPEC (enteropathogenic)		<i>Escherichia coli</i> ETEC (enterotoxic)
<i>Escherichia fergusonii</i>	<i>Escherichia hermannii</i>	<i>Gardnerella vaginalis</i>
<i>Helicobacter pylori</i>	<i>Klebsiella pneumoniae</i>	<i>Lactobacillus acidophilus</i>
<i>Proteus vulgaris</i>	<i>Providencia stuartii</i>	<i>Pseudomonas aeruginosa</i>
<i>Pseudomonas fluorescens</i>	<i>Salmonella enteric serovar minnesota</i>	
<i>Salmonella typhimurium</i>	<i>Serratia liquefaciens</i>	<i>Shigella flexneri</i>
<i>Shigella sonnei</i>		<i>Staphylococcus aureus</i>
<i>Staphylococcus aureus</i> (Cowan)		<i>Staphylococcus epidermidis</i>
<i>Yersinia enterocolitica</i>		
Human Adenovirus, Type 2, 14, 40 and 41		Human Coxsackievirus A9, B1
Human Enterovirus 69	Feline calicivirus	Human rotavirus

STRAINS/SEROTYPES

Various *E. coli* Shiga toxin-producing strains and serotypes were tested in the *SHIGA TOXIN QUIK CHEK*[™] test by both the Sorbitol MacConkey Agar (SMAC) plate and MacConkey broth culture methods. *Escherichia coli* O157 strains were also tested using CT-SMAC and ChromAgar[®] O157 plate cultures. Each strain is a clinical isolate and each was tested by a cytotoxin assay and by a polymerase chain reaction (PCR) to confirm the presence of the Shiga toxin gene(s). All organisms generated positive results for the appropriate toxin(s) when tested. Following is a list of the serotypes tested, the number of strains tested in that group type and the type of toxin produced by each strain.

Shiga Toxin Type Stx1: Strain Types - O26:H11 (5 strains), O157:H7, O111:NM (2 strains), O111a:NM, O103:H2, O103:H25, O103:H6, O103:N, O111:H11, O111:H8, O145:H16, O145:NM, O45:H2 (4 strains), O45:NM, O125:NM, O146:H21, O156:H21, O26, O5:N, O70:H11

Shiga Toxin Type Stx2: Strain Types - O26:H11, O157:H7 (4 strains), O157:NM, O8:H19 (2 strains), O8:H10, ORU:H29, O177:NM, O6:H10, O104:H4 (European 2011 outbreak strain), O121:H19 (3 strains), O121, O145:H28, O145, O113:H21, O104:H21, O55:H7, O91:H21

Shiga Toxin Type Stx1 and Stx2: Strain Types - O157:H7 (7 strains), O157:NM (2 strains), O111:H8, O111, O111:NM, O113:H21

INTERFERING SUBSTANCES (U.S. Formulations)

The following substances had no effect on positive or negative test results analyzed at the concentrations indicated: Hog gastric mucin (3.5% w/v), Human blood (40% v/v), Barium sulfate (5% w/v), Imodium[®] (5% v/v), Kaopectate[®] (5% v/v), Pepto-Bismol[®] (5% v/v), Maalox[®] Advanced (5% v/v) Steric/Palmitic Acid (40% w/v), Metronidazole (0.25% w/v), Vancomycin (0.25% w/v), Prilosec OTC[®] (5 µg/mL), TUMS (50 µg/mL), Tagamet[®] (5 µg/mL), Leukocytes (0.05% v/v), Ciprofloxacin (0.25% w/v).

PRECISION – INTRA-ASSAY

For the determination of intra-assay performance, 6 positive fecal specimens (two positive for Stx1, two positive for Stx2, two positive for both Stx1 and Stx2) and six negative fecal specimens were analyzed. Each specimen was assayed on 5 cassettes. All positives remained positive and all negatives remained negative.

PRECISION – INTER-ASSAY

The inter-assay precision of the *SHIGA TOXIN QUIK CHEK*[™] test was determined using 12 fecal specimens (six negative, two positive for Stx1, two positive for Stx2, and two positive for both Stx1 and Stx2). The samples were tested, twice a day over a 5-day period using 2 different kit lots. A positive and negative control was run on each day. All positives remained positive and all negatives remained negative.

ANALYTICAL SENSITIVITY

The cutoff for the *SHIGA TOXIN QUIK CHEK*[™] test was established at concentrations of 0.04 ng/mL Stx1 and 0.04 ng/mL Stx2.

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REFERENCES

1. O'Brien, A. D. and G. D. LaVeck. 1983. Purification and Characterization of a *Shigella dysenteriae* 1-Like toxin Produced by *Escherichia coli*. *Infect. Immun.* 40:675-683.
2. Boyce, T.G., D. L. Swerdlow, and P. M. Griffin. 1995. *Escherichia coli* O157:H7 and the Hemolytic-Uremic Syndrome. *N. Engl. J. Med.* 333:364-368.
3. Riley, L. W., R. S. Remis, S. D. Helgerson, H. B. McGee, J. G. Wells, B. R. Davis, R. J. Hebert, E. S. Olcott, L. M. Johnson, N. T. Hargrett, P. A. Blake, and M.L. Cohen. 1983. Hemorrhagic Colitis Associated with a Rare *Escherichia coli* Serotype. *N. Engl. J. Med.* 308:681-685.
4. Wong, C. S., S. Jelacic, R. L. Habeeb, S.L. Watkins, and P.I. Tarr. 2000. The Risk of the Hemolytic-Uremic Syndrome After Antibiotic Treatment of *Escherichia coli* O157:H7 Infections. *N. Engl. J. Med.* 342:1930-1936.
5. Boudailliez, B., P. Berquin, P. Mariani-Kurkdjian, D. Ilef, B. Cuvelier, I. Capek, B. Tribout, E. Bingen, and C. Piussan. 1997. Possible Person-to-Person Transmission of *Escherichia coli* O111 – Associated Hemolytic Uremic Syndrome. *Pediatr. Nephrol.* 11:36-39
6. Dundas, S., W. T. Todd, A. I. Stewart, P. S. Murdoch, A. K. Chaudhuri, and S. J. Hutchinson. 2001. The Central Scotland *Escherichia coli* O157:H7 Outbreak: Risk Factors for the Hemolytic Uremic Syndrome and Death Among Hospitalized Patients. *Clin. Infect. Dis.* 33:923-931.
7. Besser, R. E., S. M. Lett, J. T. Weber, M. P. Doyle, T. J. Barrett, J.G. Wells, and P.M. Griffin. 1993. An Outbreak of Diarrhea and Hemolytic Uremic Syndrome from *Escherichia coli* O157:H7 in Fresh-Pressed Apple Cider. *J. Amer. Med. Assoc.* 269:2217-2220.
8. Hilborn, E. D., J. H. Mermin, P. A. Mshar, J. L. Hadler, A. Voetsch, C. Wojtkunski, M. Swartz, R. Mshar, M. Lambert-Fair, J. A. Farrar, M. K. Glynn, and L. Slutsker. 1999. A Multistate Outbreak of *Escherichia coli* O157:H7 Infections Associated With Consumption of Mesclun Lettuce. *Arch. Intern. Med.* 159:1758-64.
9. Keene, W. E., J. M. MacAnulty, F. C. Hoesly, L. P. Williams Jr., K. Hedberg, G. L. Oxman, T. J. Barrett, M. A. Pfaller, and D. W. Fleming. 1994. A Swimming-Associated Outbreak of Hemorrhagic Colitis Caused by *Escherichia coli* O157:H7 and *Shigella sonnei*. *N. Engl. J. Med.* 331:579-584.
10. O'Brien, A. D., A. R. Melton, C. K. Schmitt, M.L. McKee, M. L. Batts, and D. E. Griffin. 1993. Profile of *Escherichia coli* O157:H7 Pathogen Responsible for Hamburger-Borne Outbreak of Hemorrhagic Colitis and Hemolytic Uremic Syndrome in Washington. *J. Clin. Microbiol.* 31:2799-801.
11. Mead, P. S., L. Slutsker, V. Dietz, L. F. McCaig, J. S. Bresee, C. Shapiro, P. M. Griffin, and R. V. Tauxe. 1999. Food-Related Illness and Death in the United States. *Emerg. Infect. Dis.* 5:607-625.
12. Bopp, C., and N. Strockbine. 2009. Laboratory Diagnosis of STEC Infections. ASM Teleconference.
13. Johnson, K. E., C. M. Thorp, and C. L. Sears. 2006. The Emerging Clinical Importance of Non-O157 Shiga toxin-Producing *Escherichia coli*. *Clin. Infect. Dis.* 43:1587-95.
14. Beutin, L., G. Krause, S. Zimmermann, S. Kaulfuss, and K. Gleier. 2004. Characterization of Shiga toxin-Producing *Escherichia coli* Strains Isolated from Human Patients in Germany Over a 3-year Period. *J. Clin. Microbiol.* 42:1099-108.
15. Couturier, M. R., B. Lee, N. Zelyas, and L. Chui. 2011. Shiga Toxigenic *Escherichia coli* (STEC) Detection in Stool Samples Screened for Viral Gastroenteritis in Alberta, Canada. *J. Clin. Microbiol.* 49:574-578.
16. Ethelberg, S., B. Smith, M. Torpdahl, M. Lisby, J. Boel, T. Jensen, E. M. Nielsen, and K. Mølbak. 2009. Outbreak of Non-O157 Shiga toxin-Producing *Escherichia coli* Infection from Consumption of Beef Sausage. *Clin. Infect. Dis.* 48:78-81.
17. March, S. B., and S. Ratnam. 1986. Sorbitol-MacConkey Medium for Detection of *Escherichia coli* O157:H7 Associated with Hemorrhagic Colitis. *J. Clin. Microbiol.* 23:869-872.
18. Gould, L. H., C. Bopp, N. Strockbine, R. Atkinson, V. Baselski, B. Body, R. Carey, C. Crandall, S. Hurd, R. Kaplan, M. Neill, S. Shea, P. Somsel, M. Tobin-D'Angelo, P. M. Griffin, and P. Gerner-Smidt. 2009. Recommendations for Diagnosis of Shiga toxin-Producing *Escherichia coli* Infections. *MMWR* 58:1-14.
19. Strockbine, N.A., L. R. Marques, J. W. Newland, H. W. Smith, R. K. Holmes, and A. D. O'Brien. 1986. Two toxin-Converting Phages from *Escherichia coli* O157:H7 Strain 933 Encode Antigenically Distinct toxins with Similar Biologic Activities. *Infect. Immun.* 53:135-40.
20. O'Loughlin, E. V., and R. M. Robins-Browne. 2001. Effect of Shiga toxin and Shiga-Like toxins on Eukaryotic Cells. *Microbes Infect.* 3:493-507.
21. Siegler, R.L., T. G. O'Brig, T. J. Pysher, V. L. Tesh, N. D. Denkers, and F. B. Taylor. 2003. Response to Shiga toxin 1 and 2 in a Baboon Model of Hemolytic Uremic Syndrome. *Pediatr. Nephrol.* 18:92-96.

22. Kleanthous H., H. R. Smith, S.M. Scotland, R. J. Gross, B. Rowe, C. M. Taylor, and D. V. Milford. 1990. Haemolytic Uraemic Syndromes in the British Isles, 1985-8: Association with Verocytotoxin Producing *Escherichia coli*. Part 2: Microbiological Aspects. Arch. Dis. Child. 65:722-727.
23. Louise, C. B., and T. G. Obrig. 1995. Specific Interaction of *Escherichia coli* O157:H7-Derived Shiga-Like toxin II with Human Renal Endothelial Cells. J. Infect. Dis. 172:1397-401.
24. Karmali, M. A. 1987. Laboratory Diagnosis of Verotoxin-Producing *Escherichia coli* Infections. Clin. Microbiol. News. 65-69.
25. Law, D. 2000. Virulence Factors of *Escherichia coli* O157 and Other Shiga toxin-Producing *E. coli*. J. Appl. Microbiol. 88:729-745.
26. Strockbine, N.A., M. P. Jackson, L. M. Sung, R. K. Holmes, and A. D. O'Brien. 1988. Cloning and Sequencing of the Genes for Shiga toxin from *Shigella dysenteriae* Type 1. J. Bacteriol. 170:1116-22.
27. Slutsker, L., A. A. Ries, K. D. Greene, J. G. Wells, L. Hutwagner, and P. M. Griffin. 1997. *Escherichia coli* O157:H7 Diarrhea in the United States: Clinical and Epidemiologic Features. Ann. Intern. Med. 126:505-13.
28. Bielaszewska, M., A. Mellmann, W. Zhang, R. Köck, A. Fruth, A. Bauwens, G. Peters, H. Karch. 2011. Characterisation of the *Escherichia coli* strain associated with an outbreak of haemolytic uraemic syndrome in Germany, 2011: a microbiological study. Lancet Infect. Dis. 11:671-676.

Technical Support

Further information can be obtained from contacting TECHLAB® Technical Support::

US +1 800 TECHLAB
Phone (540) 953-1664
Fax (540) 953-1665
Email ts@techlab.com

© 2023 TECHLAB, Inc. All rights reserved.

SHIGA TOXIN QUIK CHEK, the TECHLAB Logo, and TECHLAB are trademarks of TECHLAB, Inc.

All other trademarks referenced are trademarks of their respective owners.